

# HEALTH HUB

Connects you with your care team

## Plug it In

- » Plug in the Health Hub Console.

Step  
01



## Power On

- » Flip the Power Switch to the "ON" position (Located on the bottom of Console).
- » Your Health Hub will automatically connect to our cellular network.

Step  
02



## Press Call button

- » Press the lower left hand blue button to test the connection and make sure that it's properly working.

Step  
03



# BLOOD PRESSURE MONITOR

Measures your systolic/diastolic numbers

## Put the cuff around the arm

- » Slide blood pressure cuff on and velcro firmly on your bicep. Your palm should be up and the hose from the cuff should lead down the inside of your resting forearm.

Step  
04



## Press the power button

- » Press the power button on the blood pressure monitor which will inflate cuff automatically and take reading. It is best to take reading while sitting upright with feet on the floor.

Step  
05



## Turn the device off

- » Your Health Hub system will announce results. Remove cuff and turn off blood pressure monitor.

Step  
06



**YOU'RE READY  
AND PROTECTED!**



[www.nsightcare.com](http://www.nsightcare.com)