

# HEALTH HUB

Connects you with your care team

## Plug it In

- » Plug in the Health Hub Console.

Step  
01



## Power On

- » Flip the Power Switch to the "ON" position (Located on the bottom of Console).
- » Your Health Hub will automatically connect to our cellular network.

Step  
02



## Press Call button

- » Press the lower left hand blue button to test the connection and make sure that it's properly working.

Step  
03



# BLOOD GLUCOSE METER

Provides plasma-calibrated measurements

## Inserting test strip

- » Insert new test strip with black and white markings facing Blood Glucose Meter (as shown). Inserting test strip will turn meter on and meter will announce instructions.

Step  
04



## Producing blood sample

- » Use lancet to produce blood sample and place blood droplet on end of test strip.

Step  
05



## The results of your reading

- » Your meter will announce the results of your reading. Hit eject and discard test strip. Once test strip is ejected, your Health Hub will receive and repeat the reading.

Step  
06



**YOU'RE READY  
AND PROTECTED!**



[www.nsightcare.com](http://www.nsightcare.com)